

**SAMSUNG MEDISON**

# HS40

## Powered by CrystalLive™

# Data Sheet

---

**V1.05.01**

**Jan. , 2023**

**Rev 01**

**CONFIDENTIAL**

THIS DOCUMENT CONTAINS CONFIDENTIAL INFORMATION WHICH IS PROPRIETARY TO SAMSUNG MEDISON CO., Ltd. THE DOCUMENT AND ITS CONTENTS (INFORMATION) CANNOT BE COPIED OR DISCLOSED WITHOUT THE EXPRESSED WRITTEN CONSENT OF SAMSUNG MEDISON CO., Ltd.

East Central Tower, Cheonho-Dong, 1077, Cheonho-daero, Gangdong-gu, Seoul, Korea  
[www.samsungmedison.com](http://www.samsungmedison.com) / CX Group

## Table of Contents

<b>1</b>	<b>SPECIFICATION SUMMARY</b>	-----	<b>1</b>
<b>2</b>	<b>GENERAL SPECIFICATION</b>	-----	<b>3</b>
2.1	Physical Specification	-----	3
2.2	Console Design	-----	3
2.3	Main Monitor	-----	3
2.4	Control Panel	-----	3
2.5	PC	-----	3
2.6	Electrical Specification	-----	3
<b>3</b>	<b>System Specification</b>	-----	<b>4</b>
3.1	Applications	-----	4
3.2	Presets	-----	4
3.3	Operation Mode	-----	4
3.4	Display Mode	-----	4
3.5	Transducer Types	-----	5
3.6	System Standard Features	-----	5
3.7	System Options	-----	6
3.8	Display	-----	6
3.9	Language	-----	7
3.10	Operation Environment	-----	7
<b>4</b>	<b>Processing</b>	-----	<b>7</b>
4.1	Data Processing	-----	7
4.2	Pre-Processing	-----	8
4.3	Post-Processing	-----	9
<b>5</b>	<b>Connectivity</b>	-----	<b>9</b>
5.1	DICOM	-----	9
5.2	IHE	-----	10
5.3	Peripheral Interface	-----	10
<b>6</b>	<b>Scanning Parameters</b>	-----	<b>10</b>
6.1	2D Mode	-----	10
6.2	M Mode	-----	11
6.3	Color Mode	-----	11
6.4	PWD Mode	-----	11
6.5	CWD Mode	-----	12
6.6	PD Mode	-----	12
6.7	3D/4D Mode	-----	12

---

6.8	Elastoscan Mode	-----	14
<b>7</b>	<b>Transducers</b>	-----	<b>14</b>
7.1	Linear Array	-----	14
7.2	Convex Array	-----	15
7.3	Endocavity	-----	16
7.4	Volume	-----	16
7.5	Phased Array	-----	17
7.6	Pencil	-----	17
<b>8</b>	<b>Measurement</b>	-----	<b>17</b>
8.1	Caliper	-----	17
8.2	Abdomen	-----	18
8.3	Cardiac	-----	18
8.4	Carotid	-----	18
8.5	UE Artery	-----	18
8.6	UE Vein	-----	19
8.7	LE Artery	-----	19
8.8	LE Vein	-----	19
8.9	Gynecology	-----	20
8.10	Obstetrics	-----	21
8.11	Fetal Heart	-----	20
8.12	Urology	-----	20
8.13	Breast	-----	21
8.14	MSK	-----	21
8.15	Thyroid	-----	21
8.16	Testicle	-----	21
8.17	Superficial	-----	21
8.18	Pediatric	-----	21
<b>9</b>	<b>Safety / EMC</b>	-----	<b>21</b>
9.1	Classifications	-----	21
9.2	Applied Standards	-----	21
9.3	Acoustic Output Management	-----	22
9.4	Anti-virus Solution	-----	22

## SPECIFICATION SUMMARY

### PHYSICAL SPECIFICATION

- Height: 1,354 ~ 1,620mm (with monitor)
- Width: 520 mm
- Depth: 730 mm
- Weight: 54 Kg (without accessories)
- Weight: Approx. 57 Kg (with Safe Working Load)

### IMAGING MODES

- 2D-Mode
- M-Mode
- Color M-Mode
- Anatomical Mode
- Color Doppler Mode
- Pulsed Wave (PW) Spectral Doppler Mode
- Continuous Wave (CW) Doppler Mode
- Tissue Doppler Imaging (TDI) Mode
- Tissue Doppler Wave (TDW) Mode
- Power Doppler (PD) Mode
- ElastoScan Mode
- 3D/4D/XI STIC imaging Mode
- Freehand 3D Mode
- Dual Mode
- Quad Mode
- Combined Mode
- Simultaneous Mode
- Zoom Mode
- S-Flow Mode

### FOCUSING

- Transmit focusing, maximum of eight points (four points simultaneously selectable)
- Digital dynamic receive focusing (continuous)

### PROBE CONNECTIONS

- 3 or 4 Probe Connectors (Selectable)
- CW Probe Connector (Optional)

### MONITOR

- Main Monitor
  - Resolution: 1,920 x 1,080
  - 21.5 Inch LED Monitor
- Touch Screen Monitor
  - Resolution: 1,280 x 800
  - 10.1 Inch LED Monitor

### ECG

- USB Type (Type CF)

### IMAGE STORAGE

- Maximum 45,000 Frames for Cine memory
- Maximum 14,000 Lines for Loop memory
- Image filing system

### REAR PANEL INPUT/OUTPUT CONNECTIONS

- Audio Output Port (Right/Left)
- VGA monitor
- S-Video Output
- LAN
- USB Port
- HDMI Output

## AUXILIARY

---

- DVD Multi-Drive
- Digital B/W Video Printer
- Digital Color Video Printer
- USB Printer
- DVD Recorder
- Foot switch (IPX8)
- USB Flash Memory Media
- USB HDD
- USB ECG
- Monitor

## USER INTERFACE

---

- English, German, French, Spanish, Italian, Portuguese, Chinese

## ELECTRICAL PARAMETERS

---

- 100-240 VAC, 620 VA, 50/60 Hz
- Battery: 14.4VDC, 6900mAh, 99.36Watt (Optional)

## PRESSURE LIMITS

---

- Operating: 700 hPa to 1,060 hPa
- Storage: 700 hPa to 1,060 hPa

## HUMIDITY LIMITS

---

- Operating: 30 % to 75 %
- Storage & Shipping: 20 % to 90 %

## TEMPERATURE LIMITS

---

- Operating: 10 °C to 35 °C
- Storage & Shipping: -25 °C to 50 °C

## GENERAL SPECIFICATION

### PHYSICAL SPECIFICATION

- Height: 1,354 ~ 1,620mm (with monitor)
- Width: 520 mm
- Depth: 730 mm
- Weight: 54 Kg (without accessories)
- Weight: Approx. 57 Kg (with Safe Working Load)

### CONSOLE DESIGN

- 3 or 4 Active Probe Ports (Optional)
- 4 Swivel Wheel Cart Based Type
- Articulated Monitor Arm
- Built-in Printer Storages
- Ergonomic Operation Panel
- Touch Screen
- Alpha-Numeric Keyboard(Optional)
- Trackball
- Probe Holder / Gel Holder
- Front Handle
- Integrated PC Module
- Integrated SSD
- Windows10 IoT Enterprise
- ODD (Optional)
- Gel Warmer (Optional)
- ECG (Optional)

### MAIN MONITOR

- 21.5 Inches High Resolution LED Monitor
- Resolution: 1,920 x 1,080 (16:9)
- Number of Color: 16.7 M
- Brightness Adjustment
- Interactive Dynamic Software Menu

- Articulated Monitor Arm
  - Swivel: +/- 160 °
  - Tilt: + 10 ° / - 75 °
  - Lift: 180 mm

### CONTROL PANEL

- Touch Screen
  - 10.1 Inches High Resolution LED Monitor
  - Resolution: 1,280 X 800
  - Capacitive Touch Type
  - Virtual Alpha Numeric KBD
- Alpha-Numeric KBD
- 4 User Keys
- Tri-Status backlit
- 5 Probe Holders
- Height Adjustment: 100 mm

### PC

- Main Processor: Intel i3-8100H
- Main Memory: 8 GB
- Built-in SSD: 512GB

### ELECTRICAL SPECIFICATIONS

- Frequency: 50/60 Hz
- Voltage: 100 ~ 240 VAC
- Power Consumption: Max.620 VA with Peripherals
- Heat Dissipation: 2,729.7 [BTU/h]
- System Noise: under 40dBA[MAX.]
- Built-in Equipotential Circuit
- Battery (Optional)
  - Scan time: 30 min. (Basic mode Live scan)
  - Wake up time from sleep mode: About 30 sec.
  - 14.4VDC, 6900mAh, 99.36Watt Lithium-ion Battery
  - Charging time: About 5 Hours

## SYSTEM SPECIFICATION

### APPLICATIONS

- Abdomen
- Cardiac
- Gynecology
- MSK
- Obstetrics
- Pediatric
- Small Parts
- Urology
- Vascular

### PRESETS

- Abdomen
- Adult Echo
- Adnexa
- Aorta
- Aortic Arch
- Arterial
- Bladder
- Bowel
- Breast
- Carotid
- Deep
- Fetal Heart
- General
- Neo Head
- NT
- Ped Abd

- Ped Echo
- Ped Hip
- Prostate
- Renal
- Spine
- Superficial
- Thyroid
- Testicle
- TCD
- Uterus
- Venous
- 1<sup>st</sup> Trimester
- 2<sup>nd</sup> Trimester
- 3<sup>rd</sup> Trimester

### OPERATION MODE

- B-Mode (2D)
- Color Doppler Mode (C)
- Pulse Wave Doppler (PWD)
- Continuous Wave Doppler(CWD): Steered / Static
- Power Doppler Mode (PD)
- S-Flow™ Mode
- M-Mode (M)
- Anatomical M Mode
- Single/Dual/Quad Mode
- Volume Mode
  - 3D / 4D / 3D XI / XI STIC™
- TDI/TDW
- ElastoScan Mode

### DISPLAY MODE

- Dual Mode

- B+B, B+B/C, B+B/PD, B+B/S-Flow

- ElastoScan + ElastoScan

- Dual Live Mode

- B+B, B+B/C, B+B/PD, B+B/S-Flow

- B+ElastoScan

- Real-Time Triplex Mode (Simultaneous Mode)

- B+C+PW, B+PD+PW, B+S-Flow+PW, B+TDI+TDW

- Duplex, Triplex Mode

- B+C, B+M, B+3D, B+4D, B+PW, B+PD, B+S-Flow, B+CW,

B+C+PW, B+C+CW, B+C+M, B+ElastoScan, B+TDI,

B+TDW

- Quad Mode

- Combinations of B/B, B/C, B/PD and B/S-Flow,

ElastoScan

- Zoom Mode

- Write Zoom / Read Zoom / Pen zoom/ Panning

- Needle Mate+

- Panoramic

- Trapezoid

## TRANSDUCER TYPES

- Linear Array: LN5-12, L5-12/50, LA3-16AD, L4-7, LS6-15,

LA2-9S(S-Vue),

LA2-9R VET(endo linear for large animal)

- Curved Array: C2-8, C2-5, CA2-8AD, CA2-6BM,

CA1-7AD(S-Vue)

- Endo-Cavity: EVN4-9, ER4-9

- Micro-Convex Array: CF4-9, CA4-10M

- Phased Array: PN2-4, SP3-8, PA3-9B, PA1-5A

- Pencil: DP2B, DP8B

- Volume Probe (3D mechanical probe)

- Curved Volume: VN4-8

- Endo-Cavity Volume: V5-9, EV2-10A

## SYSTEM STANDARD FEATURES

- Hybrid Full Digital Beam-forming

- Frequency Range: 2 ~ 16MHz

- Displayed Imaging Depth (Probe dependent)

- Minimum Depth of Field: 2cm

- Maximum Depth of Field: 38cm

- Number of Focal Points: 1 ~ 4

- Transmission Focal Zone Position selection

- 1 ~ 8 Focal Points Selectable

(Probe and Application dependent)

- Continuous Dynamic Receive Focus / Aperture

- Multi-frequency / Wideband Technology

- Frequency Compounding (FSI)

- ClearVision

- 256 Shades of Gray

- System Internal Dynamic Range: 256

- Maximum Frame Rate

- Over 2,000 fps (Hz)

- Maximum Color Frame Rate

- Over 400 fps (Hz)

- Image Reverse: Right/Left, Up/Down

- Image Rotation: 90°, 180°, 270°

- Pre Processing

- Post Processing

- Digital Calipers / Measurement

- Cine Memory

- Capacity: 500 MB

- Cine loop: Max. 14,000 Lines

- Image storage: Max. 45,000 Frames / 60 Sec

- QuickScan™



- EzCompare
- Report Package
- Body Marker
- System Boot up: Max. 180 Sec
- Probe Change: 2-3 Sec
- User Programmable Preset : Over 30 Presets
- User Programmable Key: 4 Keys
- Touch Screen Menu Editable Function
- SonoView™
- Data Backup / Restore
- Image Exporting and Importing
- PW Velocity Range: 0.1cm/s ~ 8.8m/s
- CW Velocity Range: 1cm/s ~ 19.3m/s
- Wireless Lan
- RIS Web Browser
- Q-Path/ Q-View
- Barcode/Card Reader
- XI STIC
- MultiVision
- Strain+
- SEE Stream (RU region only)
- RUS restricted mode (RU region only)
- DEU restricted mode (DE region only)
- DVD-RW
- ECG (AHA / IEC)
- Foot Switch
- Gel Warmer
- Printer Tray (Large / Small)
- 3P connector PSA
- 3P Connector PSA (with Pencil Probe Port)
- 4P Connector PSA
- 4P Connector PSA (with Pencil Probe Port)
- StressEcho
- 2D NT

## SYSTEM OPTIONS

---

- 4D
- 3D XI™
- AutoIMT+
- Cardiac Measurement
- CW Function
- DICOM 3.0
- ElastoScan
- EzExam+
- NeedleMate+
- Panoramic+
- 5D NT
- 5D Follicle
- RealisticVue
- E-Strain
- BatteryAssist
- HQ-Vision
- 2D Follicle
- LaborAssist
- S-Detect for Breast
- CrystalVue
- Expanded Storage(1TB)
- Windows 10
- NerveTrack
- Mobile Export
- EzAssist™
- UterineAssist™

## DISPLAY

---

- Application
- Preset
  - Mode
- Date: 3 types (Selectable)
  - YYYY-MM-DD
  - MM-DD-YYYY
  - DD-MM-YYYY
- Time: 2 types (Selectable)
  - 24 hours
  - 12 hours
- Patient (General Information)
  - Patient ID
  - Patient Name (First, Middle & Last)
  - Gender: Female, Male, Other
  - Birth / Age
  - Accession Number
  - Diag. Physician
  - Ref. Physician
  - Operator
  - Indication
  - Study Information
  - E-mail
- Gestational Age: LMP/EDD/GA
- Institute
- Operator
- Probe Name
- Probe Orientation
- Depth / Width
- Focal Zone
- Focal Number
- TGC Line
- FPS (Hz)
- Frequency
- Gain
- Dynamic Range
- Map
- Frame Average
- Power
- ClearVision Index
- MultiVision Index
- Gray Bar
- Acoustic Index: TIs, Tlb, Tic
- Mechanical Index: MI
- Caliper & Measurement Result
- Indicator
- Pointer
- Body Marker
- ROI Position / ROI Size
- Wall filter
- Zoom / Panning
- Biopsy Guide Line (Probe dependent)

## LANGUAGE

---

- Display Language
  - English, French, German, Italian, Spanish, Portuguese, Chinese, Russian
- Input Language
  - English, French, German, Italian, Russian, Nordic (Norwegian, Finnish, Swedish, Danish)

## OPERATING ENVIRONMENT

---

- Temperature: 10 °C ~ 35 °C
- Humidity: Up to 90%
- Pressure: 700 ~ 1060 hPa

## PROCESSING

### DATA PROCESSING

- System Processing Channel: 860,160
- Raw Data Image Analysis
- Cine
  - Function: save / review / play / stop / pause / export /

Trim Start / Trim End

- Clipboard: displays thumbnail images of the acquired data for the current exam
- Enlarged Preview of the image
- Image Archive / Connectivity
- Image format: AVI, MPEG, JPEG, BMP, TIFF, DICOM
- Image Viewer (Sonoview™)
- Measurements, Calculations and Annotations on CINE

Playback

- Number of Image Storage : max. 350,000 images (RAW format)
- Image Preview
- Cine Image Preview
- Recalling Image from the Clipboard
- Scrolling Timeline Memory
- Start and End Frame Selections for Loop Playback

### PRE-PROCESSING

- B/M-Mode
  - Dynamic Range
  - Frame Average
  - Frequency
  - Gain

- Harmonic
- Pulse Inversion Harmonic (Probe dependent)
- Line Density
- Power
- Reject
- Scan Area
- TGC
- Write Zoom
- MultiVision (Probe Dependent)
- Beam Steering (Probe Dependent)
- Trapezoid (Probe Dependent)
- Free Angle Plane

- PW Mode
  - Filter
  - Frequency
  - Gain
  - Power
  - PRF (Scale)
  - Sample Volume Angle
  - Sample Volume Position
- CW Mode
  - Sample Rate
  - Filter
  - Gain
  - Power
  - Sample Volume Angle
  - Sample Volume Position
- Color Doppler / Power Doppler mode
  - Filter
  - Frame Average
  - Frequency
  - Gain

- Line Density

- Power

- PRF (Scale)

- Smoothing

- Sensitivity

- Steer Angle

- 3D / 4D Mode

- Scan Quality

- Volume Angle

- ElastoScan Mode

- Frame Average

- Frequency

- Line Density

- Read Zoom

- Sound

- Trace Direction

- Trace Method

- Color Doppler / Power Doppler Mode

- Balance

- Baseline

- Chroma Map

- Color Map

- Hide Color

- Invert

- Read Zoom

- 3D Mode

- 3D

- 3D XI™

- Accept ROI

- Chroma Map

- MagiCut™

- VOCAL™

- XI VOCAL™

- 5D NT

- XI STIC™

- ElastoScan Mode

- E-Gain

- Contrast

- Color Map

- Alpha Blending

- Blending Level

- Enhancement

## POST-PROCESSING

- B-Mode

- Chroma Map

- Gray Map

- Image Size

- Read Zoom

- ClearVision

- Sweep Speed

- M-Mode

- Chroma Map

- M Mode Map

- Read Zoom

- Sweep Speed

- PW / CW Mode

- Base line

- Chroma Map

- Doppler Map

- Invert

## CONNECTIVITY

## DICOM

- DICOM 3.0
- DICOM Media
- DICOM Performed Procedure Step (PPS)
- DICOM Print
- DICOM Storage
- DICOM Storage Commitment (SC)
- DICOM Structured Reporting (SR)
- DICOM Verification
- DICOM Worklist
- Gray Scale Converting
- Multi Frame
- Single Frame
- 3D Volume Frame
- Transfer Mode
  - Send after acquisition
  - Send on end exam
  - Send manually
- VOI LUT Setup

## IHE

- Scheduled Workflow (SWF)
- Patient Information Reconciliation (PIR)
- Portable Data for Imaging (PDI)
- Evidence Documents (ED)

## PERIPHERAL INTERFACE

- Audio out L/R
- D-SUB output
- S-Video output
- HDMI output

- USB 2.0 (6 ports)
- Ethernet 10/100/1000BASE-T
- Foot Switch: USB 2.0 (IPX 8)
- DVD Recorder: Sony DVO-1000MD
  - S-video, NTSC/PAL
  - Recording only
- Printers
  - Digital BW Video Printer: Sony UP-D897, Sony UP-D898MD, Sony UP-X898MD, Mitsubishi P95DW, Mitsubishi P95D
  - Digital Color Video Printer: Sony UP-D25MD, Mitsubishi CP30DW
  - USB Line Printer: Samsung CLP-620NDK, ML-2950

## SCANNING PARAMETERS

### 2D MODE

- Angle Steering: 0°, +/-7°, +/- 12°
- Chroma Map: off, 1 ~ 11
- Cine Play: On, Off
- Cine Speed: 6, 12, 25, 50, 100, 150, 200, 300
- Depth: 2cm ~ 38cm (Probe dependent)
- Dual Live
- Dynamic Range: 30 ~ 256
- Flip: L/R, U/D
- Focus Number: 1 ~ 4
- Frequency Compounding
- Frequency: 3 ~ 5 steps (Probe Dependent)
  - Pen2, Pen1, Gen, Res1, Res2
- Gain: 0 ~ 100
- Gray Map: 1 ~ 12

- Harmonic: On, Off
- Image Size: 70 ~ 100%
- Line Density: Low, Med, High
- Number of TGC Level: 8
- Frame Average: 0 ~ 9
- Power: 2 ~ 100
- Pulse Inversion Harmonic: On, Off (Probe dependent)
- QuickScan™
- Reject Level: 0 ~ 30
- MultiVision Index: Off, Low, Med, High
- ClearVision Index: Off, 1 ~ 5
- Trapezoid: On, Off (Linear Probes only)
- Scan Area: 40 ~ 100%
- Zoom
  - Read Zoom: 100 ~ 800 %
  - Write Zoom : 100%~ 1,400%
- Panning
- Free Angle Plane

## M MODE

---

- Chroma Map: Off, 1 ~ 11
- Display format
  - M-mode only
  - Up/down, Side by side
  - Size: 50/50, 70/30, 30/70
- Dynamic Range: 30 ~ 256
- Gain: 0 ~ 100
- M Mode Map: 1 ~ 12
- Power: 2 ~ 100
- QuickScan™: Off, On, Update
- Sweep Speed
- Color M

- Anatomical M

## COLOR MODE

---

- Balance: 0 ~ 16
- Baseline: -8 ~ 8
- Color Map: 1 ~ 12
- Line Density: Low, Med, High
- Dual Live: On, off
- Sensitivity: 0 ~ 5
- Frame Average: 0 ~ 5
- Frequency: 2 steps
- Gain: 0 ~ 100
- Hide Color: On, Off
- Invert: On, off
- Frame Average: 0 ~ 10
- Power: 2 ~ 100
- PRF: 0.1kHz ~ 19.5kHz (Probe dependent)
- Sensitivity: 0 ~ 5
- Smoothing: 0 ~ 5
- Steer Angle: 0°, ±15°, ±20°, ±30°
- Velocity
- Filter: 1 ~ 4
- Vel + Variance Map

## PWD MODE

---

- Auto Calc: Off, Live, Frozen
- Base Line: -8 ~ 8
- Chroma Map: Off, 1 ~ 11
- Display format: Up/down, Side by side, Doppler Only
- Display Size: 70/30, 50/50, 30/70
- Doppler Map: 1 ~ 12

- Dynamic Range: 30 ~ 256
- Frequency: 2 Steps
- Gain: 0 ~ 100
- Invert: On, Off
- Power: 2 ~100
- PRF: 1.0 ~ 22.5 kHz (Probe dependent)
- QuickScan™: On, update
- Simultaneous: On, Off
- Sound: 0 ~ 100
- Angle Correction: -80° ~ 80°
- SV Position control
- SV Size: 0.5 ~ 25mm
- Quick Angle: 0°, 60°, -60°
- Sweep Speed: 15 ~ 117 mm/s
- Trace
  - Method: Off, Mean, Max
  - Trace Direction: Both, Above, Below
- Update
- Filter: 1 ~ 4

## CWD MODE

---

- Auto Calc.: Off, Live, Frozen
- Base line: -8 ~ 8
- Chroma Map: Off, 1 ~ 11
- Display Format: Up/down, Side by side, Doppler Only
- Display Size: 70/30, 50/50, 30/70
- Doppler Map: 1 ~ 12
- Dynamic Range: 30 ~ 256
- Gain: 0 ~ 100
- Invert: On, Off
- Power: 2 ~100
- Sample Rate: 1.8kHz ~ 57kHz (probe dependent)

- QuickScan™: On, update
- Sound: 0 ~ 100
- Angle Correction: -80° ~ 80°
- SV Position Control
- Quick Angle: 0°, 60°, -60°
- Sweep Speed: 18 ~ 142 mm/s
- Trace
  - Method: Off, Mean, Max
  - Direction: Both, Above, Below
- Filter: 1 ~ 4

## PD MODE

---

- Balance: 0 ~ 16 step
- Color Map: 1 ~ 12
- Line Density: Low, Med, High
- Dual Live: On, off
- Filter: 1 ~ 4
- Frame Average: 0 ~ 5 step
- Frequency: 2 steps (probe dependent)
- Gain: 0 ~ 100
- Hide Color
- Invert: On, Off (S-Flow™ only)
- Power: 2 ~ 100
- PRF: 0.1 ~ 19.5 kHz (Probe dependent)
- Sensitivity: 0 ~ 5
- Smoothing: 0 ~ 5
- Steer Angle: 0°, +/-15°, +/-20°, +/-30°
- Filter: 1 ~ 4

## 3D/4D MODE

---

- 3D

- 4D (Live 3D)
  - Vol. Index
- Color 3D
  - Th. Low
- 3D XI™
  - Transparency
- MSV
  - MSV
  - Oblique View™
  - XI VOCAL™
- 5D NT
  - MSV
  - Layout
  - Ref. Image: A / B / C / MSV OH
- MagiCut™
  - Page
- Orientation Help
  - Init
- Curved ROI
  - Orientation Dot
- 3D Cine
  - Position
  - Bias
  - Selected Slice
  - Vol. Index
- 4D Cine
  - Rotation Angle: 30°/45°/60°/90°/180°/360°
  - Step Angle: 1°/3°/5°/15°
  - Slice Thick.
  - Ruler
- Cine Type: Volume, Image
  - Oblique View™
  - Layout
  - Play Mode: Loop, Yoyo
  - Auto Increment
  - Speed: Very Slow, Slow, Normal, Fast, Fastest
  - OVIX™
  - Trim Start, Trim End
  - Init
  - Volume Index
  - Clear Line
  - Cut Type: Line / Contour / Parallel / Plumb
- MPR
  - Image Rotation: -90° / 90° / 180°
  - Render
  - Accept ROI
  - VOCAL™
  - Solid / General / Prostate / Cystic / Sphere / Manual
  - Init
  - Layout
  - Ref. Image: A / B / C
  - Step Angle: 12° / 18° / 30°
  - Ref. Image: A/B/C/OH
  - Start
  - 3D Rotation: -90°/90°/180°
  - Pole 1 / Pole2
  - Select
  - Position
  - Bias
  - XI VOCAL™
  - Solid / Cystic / General / Manual
  - Mix



- Init
- Ref. Image: A / B / C / Ref. Contour
- Slice Direction
- Start
- Number of Slice
- Chroma Map
  - 2D Chroma Map: Map 1 ~ Map 10
  - 3D Chroma Map: Map 1 ~ Map 10
- Post Processing
  - Negative / Auto Contrast / Threshold / Sharpen / 3D CI
- Preset (Probe dependent)
  - Default / Surface / Skeleton / Extremity / Brain / User1~3
  - Load / Save / Rename / Reset
- ROI Size / ROI Position
- Rendering Preset: Default / Surface / Skeleton / Extremity / Brain / User1~3
- Scan Quality: Low, Med1, Med2, High
- Volume Angle: 10 ~ 90 (Probe dependent)
- XI STIC™
  - Scan Time (7 ~ 15 sec)
  - Trimester (1Trim, 2Trim, 3Trim)
  - Speed (Very Slow, Slow, Normal, Fast, Fastest)
  - Vol. Index
- 5D NT
- 5D Follicle
- RealisticVue
  - Light direction (9 directions)
  - Move light
  - Set color (Hue, Saturation, Lightness)
- CrystalVue
  - Mode(Context, ROI Countour, ROI Circle)

- Complexity, Strength, Transparency

## ELASTOSCAN MODE

- Line Density: Low, Med, High
- Invert: On, off
- Dual Live: On, off
- Frequency
- Gain: 0 ~ 100
- Contrast: 0 ~ 100
- Frame Average: 0 ~ 100
- Color Map: 1 ~ 5
- Alpha Blending: On, off
- Blending Level: 0 ~ 100
- Enhancement: 0 ~ 100

## TRANSDUCERS

### LINEAR

#### LA3-16AD

- Band Width : 3 ~ 16 MHz
- Radius of curvature : Flat
- Field of view : 38.4 mm
- Number of elements : 192
- Biopsy Guide : Available
- Application : Abdomen, MSK, Small Parts, Vascular, OB, GYN, Pediatric
- Safety Class: BF

#### LN5-12

- Band Width : 5 ~ 12 MHz
- Radius of curvature : Flat

- Field of view : 38.1 mm
- Number of elements : 128
- Biopsy Guide : Available
- Application : Abdomen, MSK, Small Parts, Vascular, OB, GYN, Pediatric
- Safety Class: BF

#### **L4-7**

- Band Width : 4~ 7 MHz
- Radius of curvature : Flat
- Field of view : 44.2 mm
- Number of elements : 192
- Biopsy Guide : Available
- Application : Abdomen, MSK, Small Parts, Vascular, OB, GYN, Pediatric
- Safety Class: BF

#### **LS6-15**

- Band Width: 6~ 15 MHz
- Radius of curvature : Flat
- Field of view : 25.6 mm
- Number of elements : 128
- Biopsy Guide : Not Available
- Application : Abdomen, MSK, Small Parts, Vascular, OB, GYN, Pediatric
- Safety Class: BF

#### **L5-12/50**

- Band Width : 5 ~ 12 MHz
- Radius of curvature : Flat

- Field of view : 52 mm
- Number of elements : 128
- Biopsy Guide : Available
- Application : Abdomen, MSK, Small Parts, Vascular, OB, GYN, Pediatric
- Safety Class: BF

#### **LA2-9S (S-Vue)**

- Linear Array probe
- Frequency(Center/Range) :5.6MHz/2-9MHz
- Radius of curvature : Flat
- Field of view : 44mm
- Number of elements : 192
- Biopsy Guide : Available
- Application : Abdomen, MSK, Small Parts, Vascular, Obstetrics, Gynecology, Pediatric
- Safety Class: BF

#### **LA2-9R Vet**

- Endo linear for Large animal
- Frequency(Center/Range) :6.1MHz/2-9MHz
- Radius of curvature : Flat
- Field of view : 66.56mm
- Number of elements : 128
- Biopsy Guide : Not Available
- Application : Reproduction, MSK
- Safety Class: BF

### **CONVEX**

---

#### **CA2-8AD**

- Band Width: 2 ~ 8 MHz
- Radius of curvature : 60.365 mm

- Field of view : 58 °
- Number of elements : 192
- Biopsy Guide : Available
- Application : Abdomen, MSK, OB, GYN, Pediatric, Vascular, Urology
- Safety Class: BF

#### **C2-8**

- Band Width: 2 ~ 8 MHz
- Radius of curvature : 51.07 mm
- Field of view : 68.176 °
- Number of elements : 128
- Biopsy Guide : Available
- Application : Abdomen, MSK, OB, GYN, Pediatric, Vascular, Urology
- Safety Class: BF

#### **C2-5**

- Band Width: 2 ~ 5 MHz
- Radius of curvature : 39.64 mm
- Field of view : 75 °
- Number of elements : 128
- Biopsy Guide : Available
- Application : Abdomen, MSK, OB, GYN, Pediatric, Vascular, Urology
- Safety Class: BF

#### **CF4-9**

- Center frequency : 5.65MHz
- Band Width: 4 ~ 9 MHz
- Radius of curvature : 14 mm

- Field of view : 92 °
- Number of elements : 128
- Biopsy Guide : Not available
- Application : Abdomen, MSK, OB, GYN, Pediatric, Vascular, Urology
- Safety Class: BF

#### **CA4-10M**

- Center frequency :5.65MHz
- Band Width: 4 ~ 10 MHz
- Radius of curvature : 14 mm
- Field of view : 92 °
- Number of elements : 128
- Biopsy Guide : Not available
- Application : Abdomen, MSK, OB, GYN, Pediatric, Vascular, Urology
- Safety Class: BF

#### **CA2-6BM**

- Center frequency : 3.5MHz
- Band Width: 2 ~ 6 MHz
- Radius of curvature : 20 mm
- Field of view : 86.08 °
- Number of elements : 144
- Biopsy Guide : Not available
- Application : Abdomen
- Safety Class: BF
- 

#### **CA1-7AD (S-Vue)**

- Curved Array probe
- Frequency(Center/Range) : 3.2MHz/1~7MHz
- Radius of curvature : 45.69mm
- Field of view : 70°
- Number of elements : 160

- Biopsy Guide : Available
- Application : Abdomen, Obstetrics, Gynecology, MSK, Pediatric, Vascular, Urology
- Safety Class: BF

## ENDOCAVITY

---

### EVN4-9

- Band Width : 4 ~ 9 MHz
- Radius of curvature : 10.073 mm
- Field of view : 148.092 °
- Number of elements : 128
- Biopsy Guide : Available
- Application : OB, GYN, Urology
- Safety Class: BF

### ER4-9

- Band Width : 4 ~ 9 MHz
- Radius of curvature : 10.073 mm
- Field of view : 148.092 °
- Number of elements : 128
- Biopsy Guide : Available
- Application : OB, GYN, Urology
- Safety Class: BF

## VOLUME

---

### VN4-8

- Band Width: 4 ~ 8 MHz
- Radius of curvature : 38.10 mm
- Field of view : 77.24 °
- Number of elements : 128
- Biopsy Guide : Available
- Application : Abdomen, MSK, OB, GYN, Pediatric, Vascular, Urology
- Safety Class: BF

### V5-9

- Band Width: 5 ~ 9 MHz
- Radius of curvature : 10.1 mm
- Field of view : 150.6 °
- Number of elements : 192
- Biopsy Guide : Available
- Application : OB, GYN, Urology
- Safety Class: BF

### EV2-10A

- Band Width: 2 ~ 10 MHz
- Radius of curvature : 10mm
- Field of view : 150 °
- Number of elements : 192
- Biopsy Guide : Available
- Application : OB, GYN, Urology
- Safety Class: BF

## PHASED ARRAY

---

### PN2-4

- Band Width : 2 ~ 4 MHz
- Radius of curvature : Flat
- Field of view : 90 °
- Number of elements : 64
- Biopsy Guide : Not available
- Application : Abdomen, Cardiac, Vascular, Pediatric
- Safety Class: BF

### PA1-5A

- Frequency(Center/Range) : 2.8MHz/1~5MHz
- Radius of curvature : Flat
- Field of view : 90 °
- Number of elements : 80

- Biopsy Guide : Available
- Application : Abdomen, Cardiac, Vascular, TCD, Pediatric
- Safety Class: BF

**SP3-8**

- Band Width : 3~ 8 MHz
- Radius of curvature : Flat
- Field of view : 90 °
- Number of elements : 64
- Biopsy Guide : Not available
- Application : Abdomen, Cardiac, Vascular, Pediatric
- Safety Class: BF

**PA3-9B**

- Phased Array
- Frequency(Center/Range) : 5.7MHz/3~9MHz
- Radius of curvature : Flat
- Field of view : 90 °
- Number of elements : 64
- Biopsy Guide : Not available
- Application : Abdomen, Cardiac, Vascular, TCD, Pediatric
- Safety Class: BF

**PENCIL**

**DP2B**

- Center frequency : 2.0MHz
- Application : Cardiac
- Safety Class : BF

**DP8B**

- Center frequency : 8.0MHz
- Application : Cardiac

- Safety Class : BF

**MEASUREMENT**

- Caliper
- Abdomen
- Cardiac
- Vascular
- Gynecology
- Obstetrics
- Fetal Heart
- Urology
- MSK
- Small Parts
- Pediatric

**CALIPER**

- 2D Distance
- M Distance
- 2D Trace
- 2D Trace length
- Doppler Manual Trace
- Doppler Limited Trace
- 2 Lines Angle
- 3 Points Angle
- Ellipse (Area / Circumference)
- Spline
- Open Spline
- Closed Spline
- %Stenosis (Diameter)
- %Stenosis (Area)
- 1 Distance Volume

- 2 Distance Volume
- 3 Distance Volume
- Ellipse Volume
- Ellipse + Distance Volume
- Disk Volume
- Slope
- Heart Rate (M, Doppler)
- Time (M, Doppler)
- Velocity
- Acceleration
- RI
- Volume Flow (Diameter)
- Volume Flow (Area)
- Auto Trace
- Manual Trace
- Limited Trace

## ABDOMEN

---

- Gallbladder
- Pancreas
- Bowel
- Kidney Vol. (Right / Light)
- Liver
- Spleen
- Aorta
- RA (Right / Left)
- Seg. A (Right / Left)
- Arc. A (Right / Left)
- Celiac A
- Splenic A
- Hepatic A (C / R / L)
- Hepatic V (R / M / L)

- Portal V (R / M / L)
- SMA
- IMA
- IVC
- IMV
- SMV

## CARDIAC

---

- LV (2D)
- LV Vol. (Simpson)
- LV Vol. (A/L)
- LV Vol. (Bullet)
- LV Mass
- RV (2D)
- Aorta
- LA
- LA Vol. (Simpson)
- RA
- LVOT
- RVOT
- AV
- MV
- TV
- PV
- Shunt
- IVC
- Tei Index
- Plum. Vein
- Hepatic Vein
- Tissue Doppler
- Qp/Qs
- LV (M)

- RV (M)

## CAROTID

---

- Subclavian A (Right / Left)
- CCA (Right / Left/Prox./Mid./Dist)
- Bulb (Right / Left)
- ICA (Right / Left/Prox./Mid./Dist)
- ECA (Right / Left)
- Vertebral A (Right / Left)

## UE ARTERY

---

- Subclavian A (Right / Left)
- Axillary A (Right / Left)
- Brachial A (Right / Left)
- Radial A (Right / Left)
- Ulnar A (Right / Left)
- SPA (Right / Left)

## UE VEIN

---

- Internal Jugular V (Right / Left)
- Innominate V (Right / Left)
- Subclavian V (Right / Left)
- Axillary V (Right / Left)
- Brachial V (Right / Left)
- Cephalic V (Right / Left)
- Basilic V (Right / Left)
- Radial V (Right / Left)
- Ulnar (Right / Left)

## LE ARTERY

---

- CIA (Left / Right)

- IIA (Left / Right)
- EIA (Left / Right)
- CFA (Left / Right)
- SFA (Left / Right)
- DFA (Left / Right)
- Popliteal A (Left / Right)
- ATA (Left / Right)
- PTA (Left / Right)
- Peroneal A (Left / Right)
- DPA (Left / Right)
- MPA (Left / Right)
- LPA (Left / Right)
- Metatarsal A (Left / Right)
- Digital A (Left / Right)

## LE VEIN

---

- CIV (Left / Right)
- IIV (Left / Right)
- EIV (Left / Right)
- CFV (Left / Right)
- PFV (Left / Right)
- SFV (Left / Right)
- GSV (Left / Right)
- Popliteal V (Left / Right)
- LSV (Left / Right)
- ATV (Left / Right)
- PTV (Left / Right)
- Peroneal V (Left / Right)
- MPV (Left / Right)
- LPV (Left / Right)
- Metatarsal V (Left / Right)
- Digital V (Left / Right)

## GYNECOLOGY

---

- Uterus
- Cervix
- Cyst (Right / Left)
- Ovary (Right / Left)
- Follicles (Right / Left / 1 ~ 12)
- Mass 1 ~ 3
- Ovarian A (Right / Left)
- Uterine A (Right / Left)
- Pericystic Flow
- Endometrial Flow
- Endo. Polyp
- Ovarian Mass (Right / Left)
- Uterine Tumor 1 ~ 3
- Cervical Tumor
- Ectopic Pregnancy

## OBSTETRICS

---

- Fetal Biometry
- Fetal Cranium
- Fetal Long Bone
- Fetal others
- AFI
- CTAR
- Maternal Others
- Ratio
- Umbilical Artery
- Mid Cereb A
- Uterine A (Right / Left)

- Placenta A
- Fetal Carotid (Right / Left)
- Fetal Aorta
- Renal A (Right / Left)
- Duct Venosus
- Fetal HR
- PLI

## FETAL HEART

---

- LV Vol. (Simpson)
- 2D Echo
- CTAR
- MPA
- Duct Artriosus
- IVC
- Duct Venosus
- Asc Aorta
- Dsc Aorta
- MV
- TV
- PLI
- TEI
- Fetal HR
- M Echo

## UROLOGY

---

- WG Prostate
- T-Zone Vol
- Bladder Vol.
- Residual Vol
- Renal Vol. (Right / Left)



## BREAST

---

- Mass 1 ~ 10 (Right / Left)
- Breast Flow (Right / Left)

## MSK

---

- Shoulder (Right / Left)
- Wrist (Right / Left)
- Knee (Right / Left)
- Ankle (Right / Left)

## THYROID

---

- Thyroid Vol. (Right / Left)
- Thyroid Flow (Right / Left)
- Mass 1 ~ 5 (Right / Left)

## TESTICLE

---

- Testis Vol. (Right / Left)
- Epididymis (Right / Left)
- Testis Flow (Right / Left)
- Mass 1 ~ 5 (Right / Left)

## SUPERFICIAL

---

- Superficial Vol (Right / Left)
- Superficial Flow (Right / Left)
- Mass 1 ~ 5 (Right / Left)

## PEDIATRIC

---

- Hip Angle (Right / Left)

## SAFETY / EMC

### CLASSIFICATIONS

---

- Safety
  - Type of protection against electrical shock: Class I
  - Degree of protection against electrical shock:  
Type BF Applied Part (Probes) and Defibrillation-Proof  
Type CF Applied Part (ECG)
- EMC
  - RF Emission CISPR 11: Class A
  - IEC 60601-1-2:2014 & EN 60601-1-2:2015
- Degree of protection against harmful ingress of water: Ordinary Equipment, Probes (IPX7), Foot Switch (IPX8)
- RoHS Compliant
- WEEE Compliant
- REACH Compliant

### APPLIED STANDARDS

---

- Safety & EMC
  - IEC 60601-1:2005+AMD1:2012
  - EN 60601-1:2006/A1:2013
  - ANSI/AAMI ES60601-1:2005(R)2012  
+A1:2012+C1:2009/(R)2012 +A2:2010/(R)2012
  - CAN/CSA 22.2 NO. 60601-1:14
  - IEC 60601-1-2:2014
  - EN 60601-1-2:2015
  - IEC 60601-1-6:2010+AMD1:2013
  - EN 60601-1-6:2010/A1:2015
  - IEC 60601-2-37:2007+A1:2015

- EN 60601-2-37:2008+A1:2015
- ISO 14971:2007
- Biocompatibility
  - ISO/EN 10993-1: 2009
- Labeling
  - EN 1041: 2008
  - ISO 15223-1: 2016
- NEMA/AIUM
  - NEMA/AIUM UD-2: 2004
  - NEMA/AIUM UD-3: 2004
- Block Network Port (Except DICOM communication port)
  - Ultrasound Machine allow only DICOM data through DICOM port
  - The network data of other network ports are rejected by Windows firewall
- Prohibit user from accessing windows application (such as Explorer)
  - Impossible to execute applications which is not allowed
  - Impossible to access internet web pages
- Windows Defender
  - Built-in Antivirus Solution

## ACOUSTIC OUTPUT MANAGEMENT

---

- User selectable, transducer and scanning mode dependent
- Dedicated Output Display on the system monitor display of output acoustic
- Power level, as well as thermal and mechanical indices:
- PWR – Output Power level. Range: From 2 % of maximum output
- Level is increased by 2% in each step.
- Mechanical Index (MI): 0.01~1.90 Range
- Thermal Index (TI): 0.01~6.00 Range
  - TIC – Thermal Index, Bone at Surface
  - TIB – Thermal Index, Bone at Focus
  - TIS – Thermal Index, Soft Tissue
- DICOM TLS
  - PHI transmission can be encrypted
- SSD Encryption
  - BitLocker
- Wiping Tool
  - Secure Erase for PHI Data(Support by Service Engineer only)
- Password Policy Configurability
- Menu Access Policy Configurability
- Audit Trail Log
  - All activities related to PHI access

## ANTI-VIRUS SOLUTION

---

- Disable USB Autorun Feature
  - Executable applications in USB stick are never launched
  - Prevent autorun virus through USB stick
- Dedicated Output Display on the system monitor display of output acoustic